

Operating the Adjustment Key

Note: Operating the Adjustment Key is written from the perspective of the patient with the appliance in his/her mouth.

1. To pull the lower jaw forward with the appliance in the mouth, instruct the patient to turn the Adjustment Key counter clockwise (towards the left ear). See Image 4.
2. To return the lower jaw to the starting position with the appliance in the mouth, have the patient turn the Adjustment Key clockwise (towards the right ear).

Homecare Instructions

Warning: it is imperative to use the AM Aligner each day to reduce the risk of permanent bite change.

- Rinse with water before use. Always brush your teeth and floss well before inserting the TAP 3 Elite in your mouth.
- Each morning after use, thoroughly clean your TAP 3 Elite appliance using a regular soft toothbrush, cool water and toothpaste. Hot water should not be used.
- Rinse thoroughly after cleaning and dry the appliance completely before storing in the container. It may help to leave the container open to ensure that the TAP™ 3 Elite dries thoroughly.

Warning: The TAP 3 Elite should be stored in a cool dry place. The appliance is made from sensitive materials and should not be stored where temperatures exceed 120°F, such as in the glove compartment of a car or the cargo hold of an airplane. Do not clean the appliance in hot or boiling water, nor to soak it in bleach or hydrogen peroxide which will cause the trays to distort or the lining to become brittle and delaminate.

Warning: Do not disassemble any of the TAP 3 Elite hardware. The TAP 3 Elite is a medical device and the patient must not tamper with it other than following specific instructions in the patient instruction booklet.

Note: Using hydrogen peroxide will cause the plastic materials of the appliance to deteriorate more rapidly or delaminate more easily. The best way to keep the TAP 3 Elite appliance clean is to brush it each morning after use.

Return to your prescriber at least yearly for examination and assessment to ensure that the TAP™ 3 Elite is not damaged and is still treating your snoring or sleep apnea.

Product Disposal

The TAP 3 Elite may be placed in the general household waste receptacle.

Warnings, Contraindications and Possible Side Effects

Note: Read all instructions before using the TAP™ 3 Elite

Warnings:

- This device is intended to reduce or alleviate nighttime snoring and obstructive sleep apnea (OSA). If symptoms, breathing difficulties or other respiratory disorders exist or persist, with or without the device, the patient should contact the prescriber immediately.
- The TAP™ 3 Elite should only be used as directed. Do NOT misuse the product.
- You may experience soreness or discomfort in your jaw or teeth. If discomfort persists, you should contact the prescriber.
- In the morning, you may sense a change in your bite. This change should disappear as the day progresses. If it continues with no change, you should contact the prescriber.
- US Federal Law restricts this device to sale by prescription only.
- Do NOT use if you have braces.
- Do NOT use if you have not seen a dentist in the last 12 months or are under active treatment.
- Do NOT use if you have loose teeth, loose crowns, or loose fillings.
- Do NOT use if you have temporary crowns, removable dentures, or bridges.
- You may experience obstruction of oral breathing with any oral appliance in the mouth.
- Do not store or transport the TAP™ 3 Elite outside the temperature range of -4°F to 122°F (-20°C to 50°C)
- You must have at least 8 natural, healthy teeth in each arch.
- In the event of muscle or jaw joint pain, discontinue use for a minimum of two days or until pain subsides and consult the prescriber.
- Discontinue use if experience discomfort and consult the prescriber.
- Do NOT use the TAP™ 3 Elite if vomiting or feeling nauseous.
- Do NOT use if the patient cannot remove the product themselves.
- You should return to the prescriber at least yearly, or as often as necessary, for re-evaluation.
- If the appliance becomes loose, damaged, or does not fit properly, contact the prescriber.
- Do NOT wash the TAP™ 3 Elite in the dishwasher or use dishwashing liquid to wash the TAP™ 3 Elite.
- Do NOT clean the TAP™ 3 Elite with products that contain chlorine, bleach, moisturizer, antiseptic, anti-bacterial agents or alcohol.
- Do NOT store the TAP™ 3 Elite in direct sunlight.
- The operating temperature range of the TAP™ 3 Elite is 40°F to 104°F (5°C to 40°C)
- For use by adults only.
- Discontinue use if the TAP™ 3 Elite is damaged or cracked.
- Patients who are sensitive to nickel or self-curing acrylic may experience allergic reactions. Discontinue use if reaction occurs and consult prescriber.

Contraindications for the TAP™ 3 Elite:

- This device is contraindicated for patients with loose teeth, loose dental work, dentures, or other oral conditions which would be adversely affected by wearing dental appliances.
- In addition, the appliance is contraindicated for patients who have central sleep apnea, have severe respiratory disorders or are under 18 years of age.
- Severe Class 2 or severe Class 3 malocclusion may require a custom splint.

Possible side effects:

There are possible side effects associated with using the TAP™ 3 Elite appliance. These side effects are not uncommon. If you experience any of the following side effects, you should contact your prescriber immediately.

- Slight tooth or gingival discomfort due to pressure of the appliance.
- Excess salivation initially. This will improve as you become accustomed to wearing the TAP™ 3 Elite.
- Slight jaw soreness or tightness, initially and with adjustments.
- Temporary bite change. This should subside approximately 30 minutes after the TAP™ 3 Elite is taken out of the mouth in the morning and the AM Aligner is used.
- Unconsciously taking the TAP™ 3 Elite out of his/her mouth at night.
- Orthodontic movement of the teeth.
- Pain or dysfunction of the temporomandibular joint and associated muscles.
- Permanent bite change.
- The metal parts are made of medical grade stainless steel or cobalt chromium. If the patient experiences any reaction, have him/her contact the prescriber immediately.

Clinicians Instruction for Use

TAP™ 3 ELITE with AccuTherm™



**AM AIRWAY
MANAGEMENT**

TAP SYSTEM™

PRTD105 Rev. B, 2019

myTAP



TAP Custom



myTAP PAP



TAP PAP CS



Airway Management Contacts

Manufacturer



3418 Midcourt Rd, Suite 114
Carrollton, TX 75006
Tel: (866) 264-7667
Fax: (214) 691-3151
Email: contactami@amisleep.com
Website: www.amisleep.com

United States
and Canada

Airway Management
(866) 264-7667
(214) 691-3151
contactami@amisleep.com

EU Authorized
Representative

MDSS GmbH
Schiffgraben 41
30175 Hannover
Germany
fon: +49 511 6262 8630
fax: +49 511 6262 8633

Australian
Sponsor

Emergo Australia
Level 20
Tower II, Darling Park
201 Sussex Street
Sydney, NSW 2000
Australia

Multiple Hook Sizes

There are three different hooks available for use with the TAP™ 3 Elite ; Short, Medium, and Long. The appropriate hook will be selected by the fabricator depending on the bite registration of the patient. The different size hooks allows the fabricator and the dentist the greatest range of adjustment without having to reset the hardware.


Short Hook
PART95ASM



Medium Hook
PART96ASM



Long Hook
PART97ASM

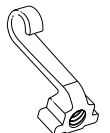

This product is protected by U.S. and foreign patents. For a list of patents related to this product please go to <http://amisleep.com/patents>

Important Safeguards

SAVE THESE INSTRUCTIONS

The following words in this manual have special significance.

Warning: Means there is a possibility of injury.

Note: Indicates point of particular interest for more efficient and convenient operation.

Indications: The Thornton Adjustable Positioner™ 3 Elite (TAP™ 3 Elite) is intended to reduce or alleviate nighttime snoring and sleep related breathing disorders including obstructive sleep apnea (OSA). The appliance is for adult patients to be used when sleeping at home or in a sleep laboratories and is for single patient use.

Contraindications: This device is contraindicated for patients with loose teeth, loose dental work, dentures, or other oral conditions which would be adversely affected by wearing dental appliances. In addition, the appliance is contraindicated for patients who have central sleep apnea, have severe respiratory disorders or who are under 18 years of age.


Intended Use

The TAP™ 3 Elite is a custom made oral device intended to reduce or alleviate nighttime snoring and obstructive sleep apnea (OSA). The appliance is for adult patients to be used when sleeping at home or in sleep laboratories and is for single patient use.

Introduction

The TAP® 3 Elite device consists of an Upper Tray that fits over the upper teeth and Lower Tray that fits over the lower teeth. A hook mechanism attached to the Upper Tray fits between two bars attached to the Lower Tray and positions the lower jaw forward, preventing the soft tissue of the throat from collapsing and obstructing the airway. The Adjustment Key of the TAP® 3 Elite permits the patient to adjust the protrusion of his/her lower jaw to the most effective and comfortable position.

The TAP® 3 Elite is supplied with the double bar design to prevent the hook from disengaging. Any adverse events that might occur due to the removal of one of the bars are the responsibility of the person performing the alteration.

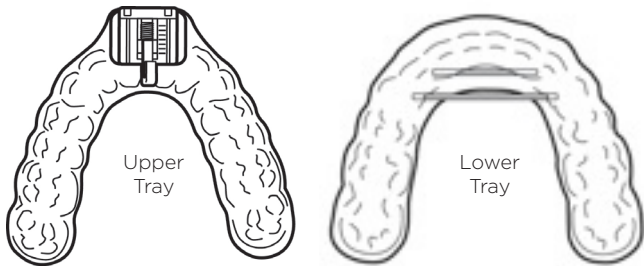

Linings: The device is fabricated on the dental casts of individual patients. The outer layer of the TAP® 3 Elite trays is a durable polymer (PET-G) and the inner layer is a thermoplastic material (AccuTherm®) which softens when heated and molded to the patient's teeth at the time of delivery. The lining is unique because it can be reheated and re-molded to the patient if the patient has had any dental work, like a crown.

Note: AccuTherm® is a heat sensitive lining and may deform if subjected to temperatures greater than 160° F. If it is heated above 160° F the impressions of your teeth in the tray will deform and your TAP® 3 Elite will no longer fit.

Each TAP™ 3 Elite package contains:

1. An Upper and Lower Tray
2. Instructions for Use
3. Storage Case
4. AM Aligner
5. Adjustment Key
6. Class I and II Hooks

Fitting the TAP 3 Elite AccuTherm Trays

Note: The following instructions only pertain to the AccuTherm™ - lined TAP 3 Elite trays.

As the patient adapts to the TAP 3 Elite, he/she should be able to sleep with it through the night. This typically takes a week.

Before you fit the patient with the TAP 3 Elite, inspect it to make sure the pieces are not damaged and are free from any physical or cosmetic defects. If there is the slightest indication that the device may be damaged or defective, do not fit it.

1. Clean the TAP 3 Elite by gently scrubbing and rinsing it thoroughly.
2. Place the Lower Tray of the TAP 3 Elite in the patient's mouth. If it does not fit immediately, easily or comfortably place the lower tray in a water bath that is heated to 160°F until the AccuTherm™ lining softens (turns mostly clear).

Warning: Do not leave the trays in the water bath too long. The hard plastic shell may warp! The trays should be removed from the water bath and fitted as soon as the lining material turns mostly clear.

3. Position the Lower Tray over the teeth. Using your thumbs, push the appliance over the teeth starting from the back and working your way forward. Press down with even pressure, especially in the posterior: do not rock appliance over the teeth.
4. Remove the Lower Tray from the mouth by the hard outer shell.
5. Once the AccuTherm has hardened or turned mostly white, place into/under cool water to speed the cooling process.
6. Place the lower tray back in the patient's mouth to check the fit.

Note: Repeat the same process (steps 2-6) for the Upper Tray.

7. Once both trays are properly fitted, have the patient place both trays in his/her mouth (they should "snap" over the teeth, but not uncomfortably). Instruct the patient to hook the trays together outside of the mouth before putting the appliance in the mouth.

Ask the patient if:

- a. the trays are tight, but not uncomfortable
- b. equal fit in all areas
- c. comfortable to the tongue
- d. he/she is able to remove the trays.

If the patient answers "no" to any of the above questions, reheat, refit, cool and seat the TAP 3 Elite appliance in the mouth until it is comfortable for the patient. With both trays in his/her mouth and hooked together, adjust the patient's jaw to a normal bite position (lips together, teeth apart and lower jaw not pulled forward) by dialing the Adjustment Key.

Note: This does not have to be exact, it is just a position that is easy to find as a starting point.

8. With the patient in this position, look at the lower unit in relation to the upper unit in the anterior area. It is extremely important that there is a minimum of 1mm space bilaterally in the posterior areas at all times, in all positions.
9. If the practitioner desires posterior support, he/she can add it once the patient reaches the final treatment position.

Warning: If any damage occurs, tell the patient not to use the appliance and return to you for repair.

Fitting the TAP 3 Elite AccuTherm Trays Cont.

If the tray is :

- **too loose:** reheat, reseat and press the trays firmly onto the dentition. Then place into/under cool water to speed the cooling process.
- **too tight:** reheat and reseat then lift the appliance up and down a few times to loosen the fit. Place into/under cool water to speed the cooling process.

Posterior Stops

It is extremely important that there is a space bilaterally between the trays prior to adding acrylic posterior stops. The objective is to create bilateral, even posterior stops at the patient's treatment position. Since the relationship of the maxilla to the mandible changes with the changes in protrusion, this procedure must be repeated if the treatment position changes.

1. Roughen the hard plastic in the occlusal area of the 1st and 2nd molar on the Lower Tray. Add auto cure acrylic to rough area.
2. Lubricate the Upper Tray with Vaseline and place in the patient's mouth. The Vaseline will keep the trays from sticking together when adding the Posterior Stops.
3. Place the lower tray in the patient's mouth while the acrylic is in the doughy stage.
4. Help the patient to couple the upper TAP™ 3 Elite tray with the lower while the lower is in the patient's mouth. Have the patient gently snap the upper tray over his/her teeth by pushing it up with his/her thumbs. Be sure the patient doesn't bite down on the stops. The Hook should be set in the patient's treatment position. See Hook Settings and Adjustments section.
5. With the trays coupled in the mouth, have the patient bite down.
6. Once the Posterior Stops are set, first smooth the area with your finger. This will ensure there isn't any rough spots that may irritate the patient.

Decreasing Lateral Movement

To decrease the lateral movement of the hook on the bar of the lower appliance, apply orthodontic acrylic under the bar. Remove acrylic in the midline to allow space for the hook and the desired lateral mobility. Place the appliance in the pressure pot or warm water bath to cure the acrylic.

Hook Setting and Adjustments

Note: The setting adjustments are written from the perspective of the practitioner looking at the patient.

The initial treatment position of the Hook should be set by the prescriber according to the following instructions. The Hook moves forward and back using the Adjustment Key to dial the Adjustment Screw clockwise or counter clockwise. Each 180 degree turn is .25 mm adjustment.

1. Instruct the patient to couple the trays together by inserting the hook between the two bars. The hook should never be engaged by hooking it onto the second (posterior) bar. Have the patient place the device in his/her mouth.
2. Dial the Adjustment Key clockwise to the patient's maximum mechanical protrusion (MMP). The patient will feel a slight stretch in his/her temporomandibular joints at this point. If the patient doesn't reach their maximum passive protrusion when engaged, continue to dial the Adjustment Key until he/she reaches this point. See Image 4.
3. Remove the trays by pulling on the posterior of the trays.
4. Place the device back in the patient's mouth.
5. Dial the Hook counter clockwise until the patient's teeth are end to end. This may be the patient's starting position.
6. Again, mark the Base Plate even with the anterior side of the Hook. This indicates the patient's starting position.

Note: On the protrusion gauge on the Base Plate, each mark and each space between the marks are 1mm wide. The protrusion gauge is 7mm long from the front of the plate to the most posterior mark. See Image 5. Have the patient wear the appliance in this end-to-end position for 3-4 nights before starting the adjustments.

7. From the starting position, instruct the patient to turn the Adjustment Key a half turn (180 degrees) counter-clockwise (towards their left ear) per night until all the symptoms are alleviated. The patient should leave the Hook in the adjusted position and not dialed back to remove the appliance. The appliance should be detached from the teeth before unhooking.
8. If any position becomes uncomfortable, the patient should dial the Hook back until pain subsides. Instruct the patient not to start dialing forward again until the jaw is comfortable. If the patient's pain does not subside, you may want to schedule a consultation for evaluation.

Note: If the patient's mandible needs to be moved further forward, there are different sizes of hooks to further range of adjustment. See section on Multiple Hook Sizes.

Note: Have the patient return to your office for examination and assessment to ensure the TAP® 3 Elite is not damaged and is still effectively treating the patient's sleep disordered breathing. If the patient loses count of how far forward the Hook is dialed, have him/her dial the Adjustment Key clockwise (towards the right ear) until the teeth are end to end (or in the starting position). Then have the patient dial the Hook as many turns counter clockwise (or forward) as the previous night's Hook position.

Directions for Daily Use

Instruct your patient in the daily use of the TAP® 3 Elite appliance. Also give your patient a copy of the Patient Instruction Booklet.

1. The patient should brush his/her teeth and floss well before inserting the TAP® 3 Elite.
2. Instruct the patient to inspect the device prior to each use. If there is any material separation, material degradation or damaged parts, the patient should discontinue use and contact you, the prescriber.
3. Tell the patient to engage the Hook of the Upper Tray with the bars on the Lower Tray before placing the trays in his/her mouth. The patient should make sure the Hook is not dialed too far forward that he/she can not comfortably snap the trays over his/her teeth while engaged.
4. The patient should gently relax once the TAP® 3 Elite is in his/her mouth.
5. After use, the patient can remove either the Upper or Lower tray by gently opening the mouth while the Hook is engaged. At the same time, the patient can lift up on the Lower Tray or pull down on the Upper Tray to loosen either tray and remove.

Warning: TAP® 3 Elite trays should never be worn separately. The patient should always wear both trays when using the appliance.

AM Aligner & Mouth Shield

The AM Aligner should be cleaned with a soft toothbrush and warm water once a week. Refer to separate AM Aligner or Mouth Shield instructions for more information.