ENGLISH

Patient Instructions for Use

Important Safeguards

SAVE THESE INSTRUCTIONS

The following words in this manual have special significance.

Warning: Means there is a possibility of injury.

Note: Indicates point of particular interest for more efficient and convenient operation.

Indications: The Thornton Adjustable Positioner[®] 3 Elite (TAP[®] 3 Elite) is intended to reduce or alleviate nighttime snoring and obstructive sleep apnea (OSA). The appliance is for adult patients to be used when sleeping at home or in a sleep laboratories and is for single patient use.

Contraindications: This device is contraindicated for patients with loose teeth, loose dental work, dentures, or other oral conditions, which would be adversely affected by wearing dental appliances. In addition, the appliance is contraindicated for patients who have central sleep apnea, have severe respiratory disorders or who are under 18 years of age.

Introduction

The Thornton Adjustable Positioner[®] 3 Elite (TAP[®] 3 Elite) is an oral device intended to reduce or alleviate nighttime snoring and obstructive sleep apnea (OSA).

The TAP[®] 3 Elite (See Image A) device consists of an Upper Tray that fits over the upper teeth and Lower Tray that fits over the lower teeth. A hook mechanism attached to the Upper Tray fits onto a lingual bar attached to the Lower Tray and positions the lower jaw forward, preventing the soft tissue of the throat from collapsing and obstructive the airway. The Adjustment Key of the TAP[®] 3 Elite permits the patient to adjust the protrusion of his/her lower jaw to the most effective and comfortable position.

The TAP[®] 3 Elite is supplied with the double bar design to prevent the hook from disengaging. Any adverse events that might occur due to the removal of one of the bars are the responsibility of the person performing the alteration.

Linings: The device is fabricated on the dental casts of individual patients. The outer layer of the TAP[®] 3 Elite trays is a durable polymer (polycarbonate) and there is a choice of two unique linings. The first lining, TAP[®] 3 Elite ThermAcryl[®] is a thermoplastic material which softens when heated and molded to the patient's teeth at the time of delivery. The lining is unique because it can be reheated and re-molded to the patient if the patient has had any dental work, like a crown.

Note: ThermAcryl[®] is a heat sensitive lining and may deform if subjected to temperatures greater than 160o F. If it is heated above 160o F the impressions of your teeth in the tray will deform and your TAP[®] 3 Elite will no longer fit.

The second lining, TAP[®] 3 Elite Triple Laminate (TL), consists of a soft inner layer that is molded to the patient's dental casts during fabrication. This lining is more comfortable than the ThermAcryl[®], but it cannot be re-molded or relined so it is not recommended for patients who are expecting to have dental work.

Each TAP[®] 3 Elite package contains:

1. An Upper and Lower Tray
2. Instructions for Use
3. Storage Case
4. AM Aligner
5. Adjustment Key

Note: The Double Lingual Bar design (see Image A) is required for Medicare billed devices. The modified design was created to meet Medicare's requirements to "lock" or "hinge" the devices together. Refer to Figure 1 for an image of the TAP[®] 3 Elite Double Lingual Bar design.

Warnings and Possible Side Effects

Note: Read all instructions before using the TAP[®] 3 Elite

Warnings:

- This device is intended to reduce or alleviate nighttime snoring and obstructive sleep apnea (OSA). If symptoms, breathing difficulties or other respiratory disorders exist or persist, with or without the device, the patient should contact the prescriber immediately.
- You may experience soreness or discomfort in your jaw or teeth. If discomfort persists, you should contact the prescriber.
- In the morning, you may sense a change in your bite. This change should disappear as the day progresses. If it continues with no change, you should contact the prescriber.
- You may experience obstruction of oral breathing with any oral appliance in the mouth.
- You should return to the prescriber at least yearly, or as often as necessary, for re-evaluation. If the TAP[®] 3 Elite is lined with ThermAcryl[®], the lining will need to be replaced at least yearly. If the appliance becomes loose, damaged, or does not fit properly, contact the prescriber.
- Allergic reactions may be encountered in people who are sensitive to nickel or self-curing acrylic.

Possible side effects:

- There are possible side effects associated with using the TAP[®] 3 Elite appliance. These side effects are not uncommon. If you experience any of the following side effects, you should contact your prescriber immediately.
 - Slight tooth or gingival discomfort due to pressure of the appliance.
 - Excess salivation initially. This will improve as you become accustomed to wearing the TAP[®] 3 Elite.
 - Slight jaw soreness or tightness, initially and with adjustments.
 - Temporary bite change. This will subside approximately 30 minutes after the TAP[®] 3 Elite is taken out of the mouth in the morning and the AM Aligner is used.
 - Unconsciously taking the TAP[®] 3 Elite out of his/her mouth at night.
 - Movement of the teeth.
 - Pain in the jaw joint
 - Permanent bite change.

Directions for Daily Use

1. Inspect the device prior to each use. Contact your prescriber if you observe any material separation, material degradation or damaged parts.
2. Engage the Hook of the Upper Tray between the two bars before placing the trays into your mouth. Make sure the Hook is not dialed too far forward that you cannot comfortably snap the trays over your teeth while engaged.
3. Gently relax your jaw once the TAP[®] 3 Elite is in your mouth.
4. To remove either the Upper or Lower Trays, gently open your mouth. Lift up on either the Upper or Lower Tray on one side from the back of the mouth, next to the last tooth.

Warning: The TAP[®] 3 Elite trays should never be worn separately. Always wear both trays when using the appliance.

Warning: The Hook should never be engaged by hooking it onto the second (posterior) bar.

Operating the Adjustment Key

Note: Operating the Adjustment Key is written from the perspective of the patient with the TAP[®] 3 Elite is in his/her mouth. Always adjust your appliance when looking in the mirror.

1. To pull your lower jaw forward with the appliance in the mouth, turn the Adjustment Key counter clockwise (towards your left ear). See Figure B.
2. To return the lower jaw to the starting position with the appliance in your mouth, turn the Adjustment Key clockwise (towards your right ear).

Note: One turn of the key is 180 degrees or .25 mm.

AM Aligner

Refer to separate AM Aligner Instructions.

Home Care Instructions

Warning: It is imperative to use the AM Aligner each day to reduce the risk of permanent bite change.

- Each morning after use, thoroughly clean your TAP[®] 3 Elite appliance using a regular soft toothbrush, cool water, and an antibacterial liquid soap or ADA approved toothpaste. Do not use hot water.
- Rinse thoroughly and dry your appliance completely before storing in the container. It may help to leave the container open to ensure that your TAP[®] 3 Elite dries thoroughly. Please store your appliance out of reach of pets and small children.
- You may freshen the TAP[®] 3 Elite every 2 weeks in a solution of diluted hydrogen peroxide (use equal parts hydrogen peroxide solution and water) for no more than 5 minutes. Afterwards, the appliance must be rinsed thoroughly with cool water and allow to dry. Always brush your teeth and floss well before inserting the TAP[®] 3 Elite in the mouth.

Note: Using hydrogen peroxide will cause the plastic materials of the appliance to deteriorate more rapidly or delaminate more easily. The best way to keep the TAP[®] 3 Elite appliance clean is to brush it each morning after use. Sparingly soak the appliance if further measures need to be used in order to clean the appliance.

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